


TrachFlush



**A NOVEL AND EASE-OF-USE MEDICAL DEVICE FOR
INVASIVE MECHANICAL VENTILATED PATIENTS**

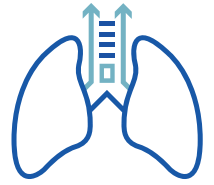

TRACHFLUSH CAN **REDUCE VENTI-
LATOR- ASSOCIATED PNEUMONIA**



TRACHFLUSH CAN **ELIMINATE
THE NEED FOR MANUAL TRACHEAL
SUCTIONING** AND THEREBY
REDUCE WORKLOAD AND
IMPROVE PATIENT COMFORT



ITALIAN INVENTION



DANISH DESIGN

TRACHFLUSH

1 DEVICE WITH 2 FEATURES

CUFF CONTROL AND FLUSH CONTROL

CUFF CONTROL

TrachFlush automatically maintains the user set cuff pressure, blocking subglottic secretions from entering the lungs, and reduces VAP with 50% ⁽²⁾.

FLUSH CONTROL

TrachFlush automatically removes secretion from invasive mechanically ventilated patients with the push of a button ⁽¹⁾.




BENEFITS OF USING TRACHFLUSH



Improving patient comfort

No "vacuum" cleaner in nor obstruction of the airways. No post-suctioning patient instability. ^{(4) (5)}

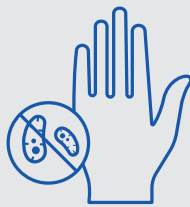


Reduces contamination risks for staff

Remove secretion without being near the patient.



VAP reduction

Reduces VAP with 50%.⁽²⁾

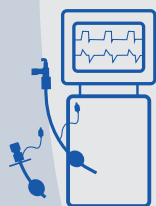

Ease of Use

Use TrachFlush with any mechanical ventilator, ETT and TT – simple connectivity.



Cost effective

Remove secretion in less than 1 minute, reduce workload and consumable usage.



Fast and easy activation


Secretion removal with the push of a button.⁽¹⁾

CUFF CONTROL

TRACHFLUSH CAN REDUCE VENTILATOR-ASSOCIATED PNEUMONIA


TrachFlush automatically maintains the user set cuff pressure, blocking subglottic secretions from entering the lungs, and reduces VAP with 50% ⁽²⁾.

BENEFITS

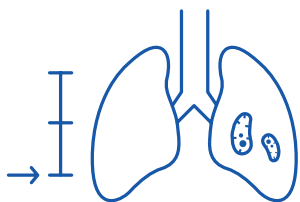

VAP reduction

Reduces VAP with 50%. ^{(2) (3)}

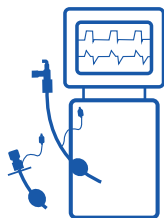

Ease of Use

Use TrachFlush with any mechanical ventilator, ETT and TT – simple connectivity.



Cost effective

Maintains cuff pressure during mechanical ventilation, reduces workload and VAP costs. ^{(2) (3) (7)}

TRACHFLUSH CUFF CONTROL STEP-BY-STEP


1

Set the cuff pressure by pressing arrow up or down on TrachFlush.




2

Accept the cuff pressure by pressing the accept button.

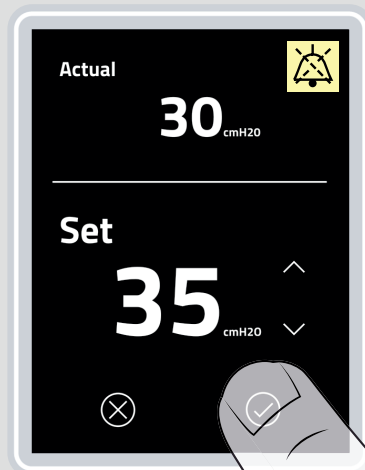

TrachFlush automatically maintains the user set cuff pressure.

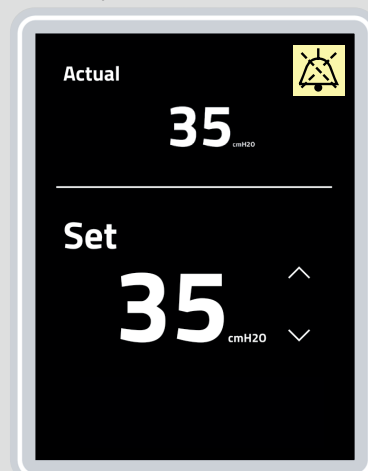

FLUSH CONTROL

**TRACHFLUSH CAN ELIMINATE
THE NEED FOR MANUAL
TRACHEAL SUCTIONING AND
IMPROVE PATIENT COMFORT**

TrachFlush automatically removes secretion from invasive mechanically ventilated patients with the push of a button.⁽¹⁾

**IMPROVE PATIENT COMFORT
WHEN USING TRACHFLUSH**



NO PATIENT DISCOMFORT (4)(5)(6)


NO OBSTRUCTION OF THE AIRWAYS (4)(5)(6)

NO POST-SUCTIONING INSTABILITY (4)(5)(6)

NO CONTAMINATION RISK FOR STAFF


TRACHFLUSH FLUSH CONTROL STEP-BY-STEP

Clinical indications for secretion removal.

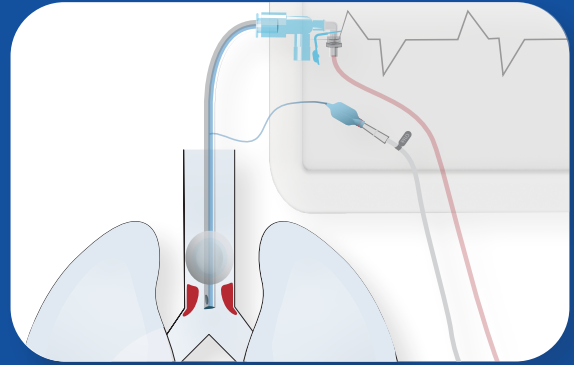


1

Nurse adjust ventilator settings if needed and if safe for the patient.

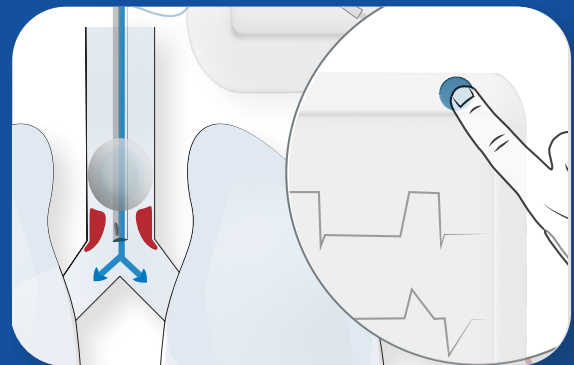


2

Nurse presses the "Flush" button. *TrachFlush deflates the cuff in alignment with the ventilator inspiratory pressure in the lungs, allowing the air-flow of pressure to push/flush the secretion from below the cuff and all the way up into the oral cavity.*

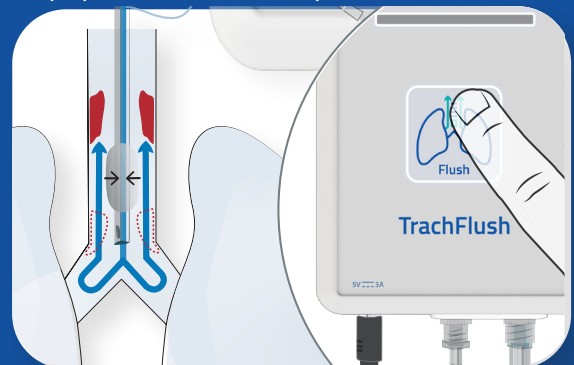


3

After flush of the secretion, the cuff is automatically inflated again within the inspiratory cycle.

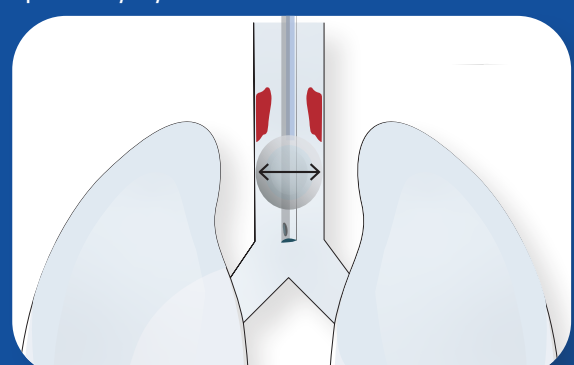

TRACHFLUSH CAN ELIMINATE THE NEED FOR MANUAL TRACHEAL SUCTIONING AND REDUCE WORKLOAD

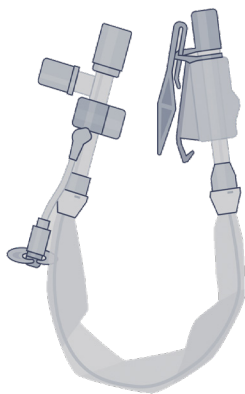

TODAY'S SUCTIONING PROCEDURE

3 MINUTES

Deliver 100% oxygen to the patient prior to suctioning, prepare vacuum source, suctioning catheter, trap, gloves, goggles, mask and other appropriate equipment and pre-oxygenate with 100% oxygen to the patient prior to suctioning.

2 MINUTES

Insert catheter and start suctioning. Monitor oxygen saturation, check for airway clearance indications and repeat if necessary. Dispose the suction catheter or rinse it. If the patient is uneasy/stressed, calm down the patient and await stability prior to repeat of suctioning.

Hyper-oxygenation of the patient. If the patient is uneasy/stressed post-suctioning, calm down the patient and await stability. Monitor the patient for adverse reactions.

SECRETION REMOVAL INDICATIONS



PATIENT & RESOURCE PREPARATION



SUCTIONING PROCEDURE



POST-SUCTIONING FOLLOW-UP CARE



TRACHFLUSH PROCEDURE

NONE

Check that the consumables are properly connected.

1 MINUTE

Press the "Flush" button. Check for airway clearance indications and repeat if necessary.

Monitor the patient for adverse reactions.

UP TO 80 % WORKLOAD REDUCTION WHEN USING TRACHFLUSH

Reference: ARC Clinical Practice Guidelines: Endotracheal Suctioning of Mechanically Ventilated Patients With Artificial Airways 2010

Secretion removal indications: Clinical indications based on patient status or guidelines. **Resources/Equipment preparation:** A licensed or credentialed respiratory therapist who have the skills and training should perform the suctioning. Check if vacuum source and suction regulator functions properly; a collection/trap bottles is connected to airway system; gloves, goggles, mask and other appropriate equipment is equipped; and prepare manual resuscitation bag with oxygen-enrichment for emergency use. **Suctioning procedure:** Insert catheter, start suctioning, monitor patient for oxygen saturation. If closed suction; leave catheter with patient. If open suction; dispose catheter. **Post-suctioning follow-up care:** Hyper-oxygenate the patient with same techniques to pre-oxygenate. Monitor for adverse reactions.




AW Technologies ApS

Amalienborgvej 57
9400 Noerresundby
Denmark
(+45) 7777 4048

info@awtechnologies.dk
www.trachflush.com

DISTRIBUTED BY :

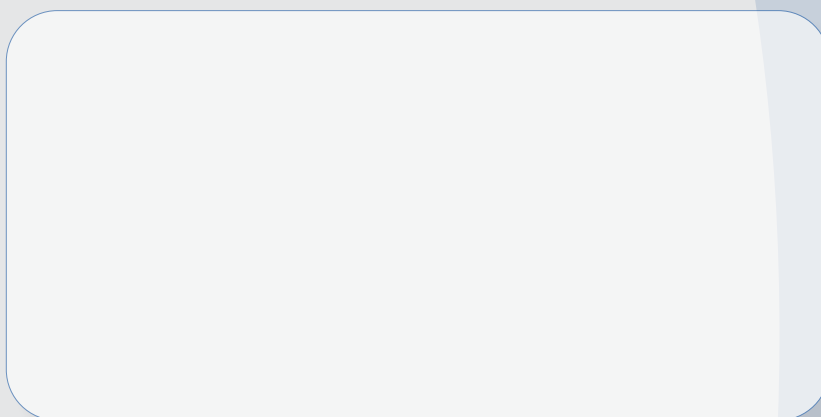

References:

1. An Artificial Cough Maneuver to Remove Secretions from Below the Endotracheal Tube Cuff by Antonio M Pesenti et al.
2. Continuous Endotracheal Tube Cuff Pressure Control System Protects Against Ventilator-Associated Pneumonia by Leonardo Lorente et al.
3. Continuous control of tracheal cuff pressure for VAP prevention: a collaborative meta-analysis of individual participant data by Saad Nseir et al.
4. Patients' recollections of experiences in the intensive care unit may affect their quality of life by Cristina Granja et al.
5. Decreasing the Adverse Effects of Endotracheal Suctioning During Mechanical Ventilation by Changing Practice by Salvatore Maggiore et al.
6. Endotracheal tube management during mechanical ventilation: less is more! By Robert M. Kacmarek et al.
7. Determining the Ideal Strategy for Ventilator-associated Pneumonia Prevention. Cost-Benefit Analysis by Westyn Branch-Elliman et al.



ITALIAN INVENTION



DANISH DESIGN