


# TrachFlush Quick Guide

## Power ON / OFF

Press 'n' hold 3 sec

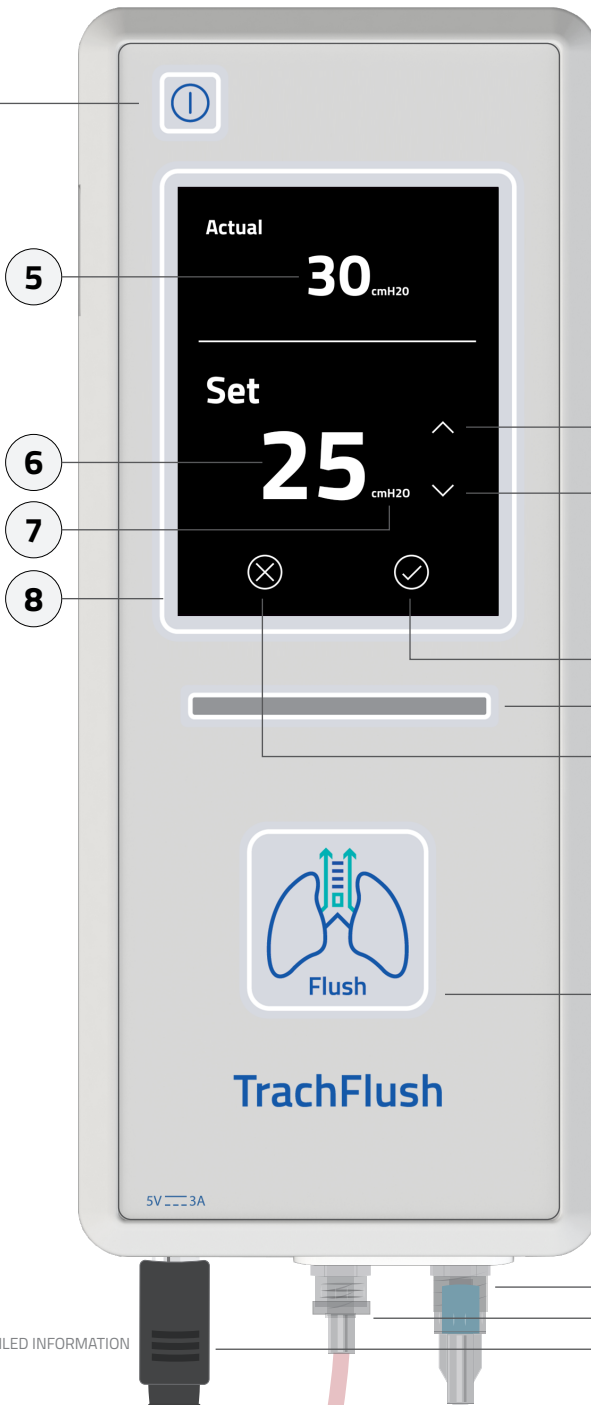

**1 Up**  
Adjust cuff pressure up

**2 Down**  
Adjust cuff pressure down

**3 Accept**  
Accept cuff pressure changes

**4 Decline**  
Decline cuff pressure changes

**5 Actual Value**  
Current cuff pressure

**6 Set Value**  
Target cuff pressure


**7 Unit of Measure**

**8 Main Screen**

## ALARM APPEARANCE

Alarms appear in the marked area on the main screen

### Alarm Silence Button

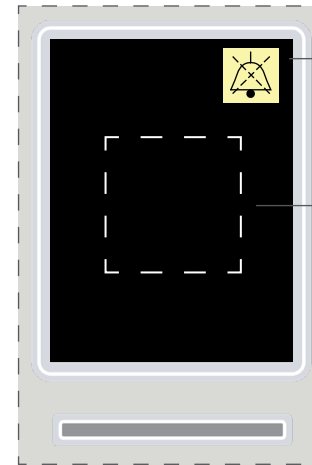

### LED Bar

When an alarm is triggered, the LED lights up with a corresponding frequency and color.

## Flush Button

Press 'n' hold 3 sec

Cuff Pressure Tube  
Airway Tube  
Power Supply

## ALARMS



### Cuff Pressure Extremely High

Cuff pressure  $\geq 70$ cm H<sub>2</sub>O



### Cuff Pressure High

Cuff pressure  $> 50$ cm H<sub>2</sub>O



### Cuff Pressure Low

Cuff pressure  $< 5$ cm H<sub>2</sub>O



### Cuff Regulation Error

Change ETT/TT or detach TrachFlush



### Power Failure

Power is disconnected

## TECHNICAL ALARMS



### Cuff Pressure Tube Not Connected

Connect Cuff Pressure Tube



### Airway Tube Not Connected

Connect Airway Tube



### Flush Not Possible

Check ventilator and cuff pressure settings



### User Shutdown Denied

Flush or cuff adjustment in progress

## INFORMATION SIGNALS



### Flush Complete



### Flush In Progress



### Processing

Checking ventilator and cuff settings



### Service

Service of the device is required



### Booting

Device starting up